


SARS-CoV-2

Antigen Rapid Test Kit

(Colloidal Gold Immunochromatography)

CE

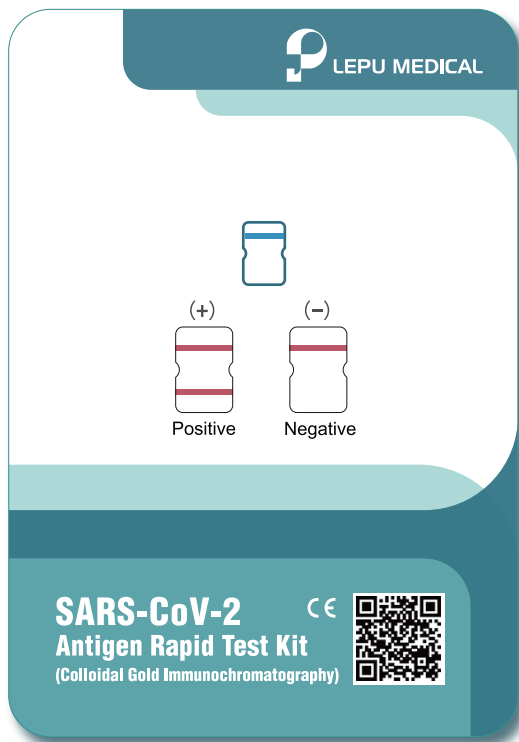

- ① Non-invasive
- ② Simple to use
- ③ Convenient, no devices required
- ④ Rapid, get result in 15 minutes
- ⑤ Stable, with high accuracy
- ⑥ Inexpensive, cost-efficiency



Hospital



Test Site



Airport



Station



Hotel




Corporation




Mass
Screening




Instruction

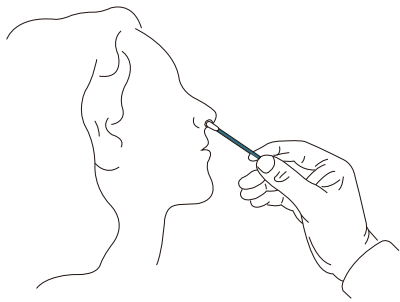


Step 1: Use swab to collect sample.

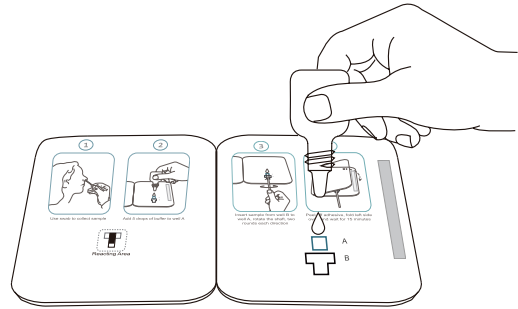

Step 2: Add 5 drops of buffer to well A.

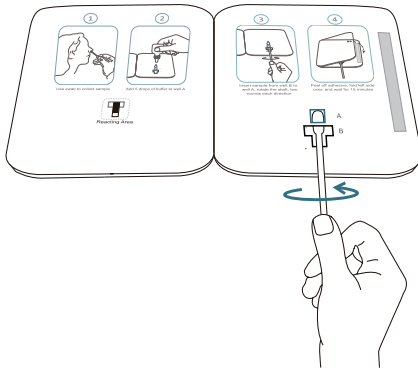

Step 3: Insert the sample through well B to well A, rotate the shaft clockwise first and counterclockwise, twice each.

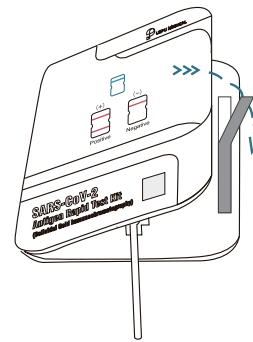

Step 4: Peel off the adhesive, fold the left side over, and wait for 15 minutes.



Result Interpretation


Positive

(+)




Negative

(-)

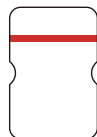

Invalid

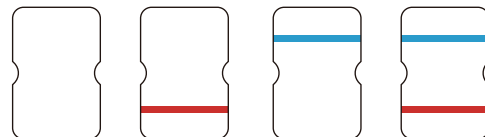

ROMAMEDICARE
E: SALES@ROMALIMITED.COM
HTTP://WWW.ROMALIMITED.COM
T: +44 (0) 1628 784966
M: +44 (0) 7709108758